

# WARDRAY PREMISE

## MR5502 MR Conditional Adjustable Height Patient Trolley



## OPERATORS MANUAL



## Introduction

Thank you for purchasing this MR Conditional product. Manufactured and tested to the highest standards it is guaranteed MR Conditional to 7.0T.

Our products are manufactured by Wardray Premise Ltd to BS EN ISO 13485 and are CE marked.

### **Manufacturing address:**

**Wardray Premise Limited, Northern Regional Office,  
Unit 2, Springvale Works, Elland Road,  
Brighouse, West Yorkshire, HD6 2RN, UK**



To ensure that you obtain maximum benefit from your product, please take a few minutes to read the enclosed information regarding operation, service and maintenance.

If you have any problems in the meantime or would like any advice about this or any other MR products from the Wardray Premise range, please contact us at our **Head Office**:

**Wardray Premise Limited, Unit 2 Mulgrave Chambers  
26-28 Mulgrave Road  
Sutton, Surrey, SM2 6LE, UK**

**Tel: +44 (0) 20 8398 9911  
Fax: +44 (0) 20 8398 8032**

E-mail: [sales@wardray-premise.com](mailto:sales@wardray-premise.com)

### **Kindly note:**

If you modify a Wardray Premise product you will invalidate your warranty.  
Unless you have our express written permission to modify a product, we transfer all liabilities for modified products to you.

### **EU Authorised Representative:**

Advena Ltd  
Tower Business Centre  
2<sup>nd</sup> Floor, Tower Street  
Swatar  
BKR 4013  
Malta



## **Safety Definitions for MRI as Defined by International Standard F2503**

### **MR SAFE:**



An item that poses no known hazards resulting from exposure to any MR environment. MR SAFE items are composed of materials that are electrically non-conductive, non-metallic, and non-magnetic.

### **MR CONDITIONAL:**



An item with demonstrated safety in the MR environment within defined conditions. At a minimum, address the conditions of the static magnetic field, the switched gradient magnetic field and the radiofrequency fields. Additional conditions, including specific configurations of the item, may be required.

Supplementary marking – additional information that, in association with marking as “MR CONDITIONAL,” states via additional language the conditions in which an item can be used safely within the MR environment.

### **MR UNSAFE:**



An item which poses unacceptable risks to the patient, medical staff or other persons within the MR environment.

## Specification

The MR5502 Trolley is designed to be MR Conditional and is manufactured and tested to the highest standards.

The MR5502 is constructed from 3 main materials, these being aluminium, brass and 316 stainless steel.

The Top Frame, Pivot Sections, Columns and Base Frame are all manufactured from aluminium, which whilst being MR compatible is also lightweight, this aids the easy movement of the trolley.

Brass and stainless steel materials are used in the manufacture of the Pivot Bolts and parts within the Hydraulic Pump and Cylinder.

It is advisable that the product is inspected and serviced every 12 months by a Wardray Premise trained engineer.

A service contract package is available, and details are available upon request from our Service Department on +44 (0) 20 8398 9911.

For international customers please contact your local Wardray Premise representative for assistance.

<b>Table top size mm 1930 x 592</b>	<b>Max length mm 2010</b>	<b>Max width mm 785 (cot-sides down)</b>	<b>Low height mm 625 (table-top)</b>	<b>Top height mm 925 (table-top)</b>	<b>Max load Kg 300</b>
---	-----------------------------------	--	--	--	--------------------------------


**Maximum patient weight 47 Stone / 300 Kg**



## Important Information


This MR Conditional trolley has been designed for adult patients. If you use the trolley to transport a child extra care should be taken particularly regarding the movement of the child across the mattress and the potential for their limbs or other body parts to protrude through the trolley sides, potentially causing injury / harm. Wardray Premise Ltd also offer a range of paediatric trolleys with cot bumpers for the safe transportation of children.

## MR5502 Trolley parts reference

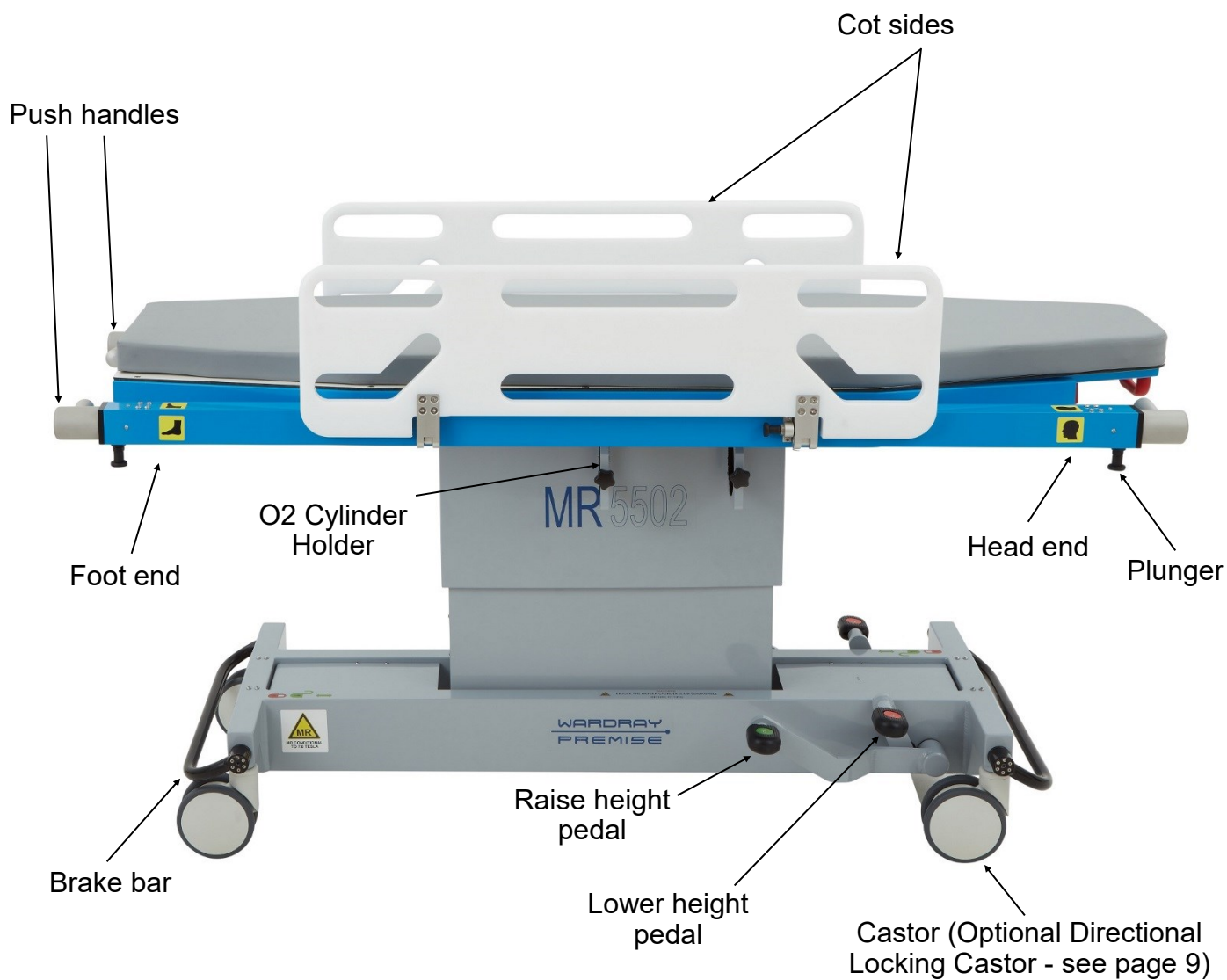

Backrest gas cylinder

Lever to operate backrest

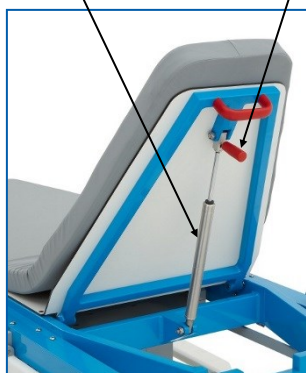

**Trendelenburg position**

Trendelenburg handle



## Operating Instructions

### **Raising and Lowering Trolley**

There are two pedals located each side of the trolley. The green pedal (Figure 1) is the pedal that raises the height, and the red pedal decreases the height.

To raise the trolley place your foot on the black and green rubber (Figure 1) and push down until you feel it stop, lift foot and allow the pedal to raise until it stops, repeat until desired height is reached, or top height is achieved.

To lower, place foot on black and red rubber pedal and push down until you feel the stop, then push further and table top will lower (Figure 2).

Remove foot when table reaches desired height or lowest position is achieved.



Fig 1



Fig 2



## Operating Instructions

### Trendelenburg

The Trendelenburg can be operated easily by placing your hands on the push bar, at the head end, and pulling the handle up towards you with your fingers as shown (Fig 1). Push the top down to the desired angle (Maximum of 13 degrees - Fig 2) and release handle. To bring the top to a horizontal position, pull the handle which will then return the top to the horizontal position.

**Please note:-** when returning to the horizontal position, the upward movement is power assisted.

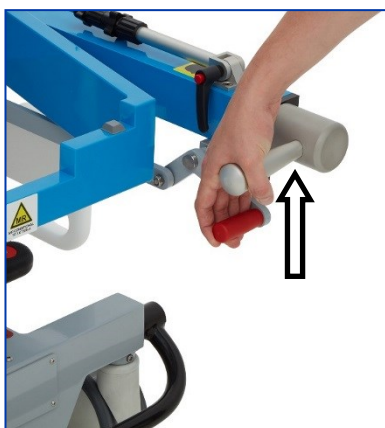


Fig 1



Fig 2

### Backrest

The backrest is operated by the handle located under the backrest board on the head end of the trolley, which is indicated via the head and foot stickers at either end of the trolley. The backrest movement is from 0 to 65 degrees, this is controlled by a gas spring.

To adjust the backrest, pull up on the **red** lever and raise the backrest into the desired position (Figure 1). Let go of the lever to lock it into that position.

When moving the backrest into its flat/horizontal position, ensure nothing can become trapped between the framework and the backrest.



Fig 1

**IMPORTANT NOTE:** When adjusting the backrest position while a patient is on the trolley we strongly advise this is done by two members of staff, one to support the patient and one to adjust the backrest.

## Operating Instructions

### Cot Sides

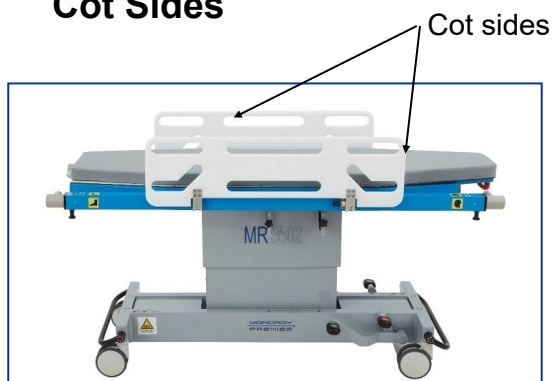


Fig 1



Fig 2



Fig 3

The trolley is fitted with raise and lower cot sides (Fig 1) these protect the patient during transportation to and from the ward.

The cot sides should not be used to steer/ push the trolley.

**When lowering cot sides ensure nothing can become trapped between the cot sides and or the bed.**

To lower the cot side, unlatch the plunger as shown in Figure 2, and then it can be folded down to the side.

To fix the cot sides into the upright, rotate the Cot Sides (Figure 3) until it is in the upright position. When the cot side is raised the plunger will latch back and hold it upright. .

**IMPORTANT NOTE: Do not attempt to push the cot sides down while the plunger is still engaged.**



## Operating Instructions

### Castors



Fig 1



Fig 2



Fig 3

The trolley is fitted with four 150mm diameter, central breaking castors. The breaking and directional lock features are controlled by the break bar (Fig 1).

In the central (middle) position the breaks are in the free-spin mode.

To engage the brakes, place your foot on the break bar (Fig 1) and push down firmly, the brake will lock in place and stop all four castors from moving.

To release the brake place your foot underneath the break bar (Fig 2) and lift it into the middle position.

The MR5502 has 1 directional locking castor to aid transportation and improve manoeuvrability. To enable the directional locking castor, place your foot underneath the break bar and lift it to the raised position (fig 3).

## Operating Instructions

### Oxygen Cylinder Holder

An Oxygen Cylinder holder is also optional, suitable for a size E cylinder, it is usually fitted underneath the blue top frame (shown in Fig 1).

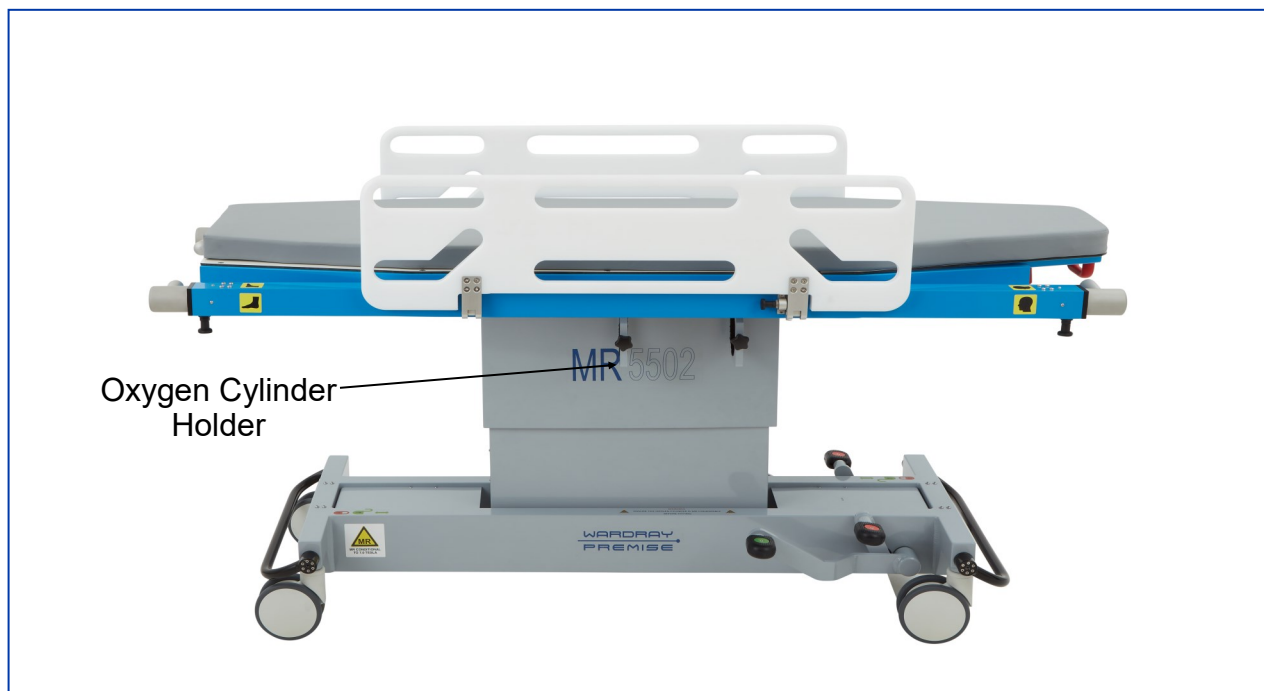


Fig 1

**WARNING:** THE OPERATOR MUST ENSURE THE OXYGEN CYLINDER IS SAFE TO USE IN YOUR MR ENVIRONMENT.

### IV Pole

An IV Pole is provided with the trolley, and is located and stored on the Oxygen Cylinder Holder (Fig 2). There are 2 IV Sockets on the trolley, one at each end, (fig 3)

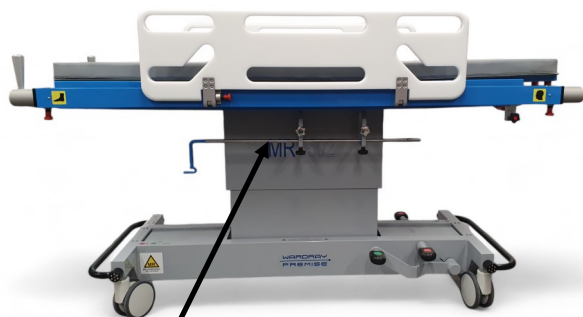


Fig 2

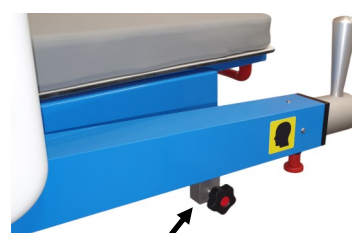


Fig 3

Bracket

## Do's and Dont's

### **Do ensure the following are observed:**

- The trolley should only be used by properly trained staff at all times.
- Ensure the Cot Sides are in an upright position before raising and lowering
- The trolley must be operated by 2 people when transporting a patient.
- If the persons operating the patient trolley experience any severe pain stop pushing immediately and consult a healthcare professional.
- When a patient is on board, the cot sides must be in an upright locked position before moving the trolley or adjusting the trolley height.
- The trolley must only be used for the purpose for which it was designed.
- When disposing of the trolley, ensure that the hydraulic oil is drained and removed, and disposed at an oil bank. In the UK find the nearest oil bank to you at: <http://www.oilbankline.org.uk/>. For outside the UK check your governments guidance and dispose of the oil appropriately.

### **Don't:**

- Lift the trolley upwards by the top
- Push the trolley over rough or uneven ground
- Drop heavy or sharp objects onto the solid plastic top
- Push the trolley by the cot sides, these are for patient safety only
- Place a weight of over the equivalent of 34 Stone (300Kg/660lbs) onto the trolley.
- Spill oil or other liquids/chemicals on the mattress
- Attempt to rectify any faults on the trolley without contacting Wardray Premise Ltd
- Use the trolley without a mattress
- Do not allow staff or patients to fix or store any items on the trolley unless you have verified the MR safety of the items

The trolley should be serviced regularly, but only by Wardray Premise trained engineers, as they will carry the correct spares to guarantee safety and compatibility of the trolley. Failure to comply with this will invalidate your warranty and could compromise its MR compatibility and therefore safety.

Any serious incidents that occur in relation to this device should be reported to Wardray Premise LTD and the competent authority of the member state in which the user and/or patient are established.

## General Maintenance

It is strongly recommended that the following checks are carried out on a weekly basis by trained personnel, to ensure trouble free operation of the trolley.

- Check the solid plastic top for cracks or other obvious signs of damage or contamination. Report any damage found.
- Check mattress for any sign of damage or contamination. Report damage or clean where necessary with mild soap and water.
- Check castors for free movement and the braking mechanism for functionality, check and remove any debris that may have been caught between the forks and around the axles. This is a common occurrence and can lead to a shortened life expectancy of the castor if left unattended.
- Check the backrest function.
- Check the Trendelenburg function of the trolley.
- Particular attention must be made to the raise and lower function. Raise and lower the trolley a couple of times before using, check for any leaks, especially if the trolley has not been used for a long period of time.
- Check the cot sides for functionality and damage, especially the cot side brackets and plungers, as these take the brunt of any impacts from the side. Ensure that the plungers work freely and locate into the holes.
- Ensure that the IV Pole fitted is correct and the serial no. corresponds with the trolley serial no. located on the inside of the backrest board (see page 5 for location). Make sure there are no sharp burrs from misuse.

Please register your product with us, via the Warranty Registration Form (see page 15), for ease of contact in case of urgent upgrades or recalls.

**If the unit is damaged please do not attempt to repair as this could invalidate your warranty and could compromise its MR integrity and therefore safety, please call or email Wardray Premise Limited:**

T: +44 (0) 20 8398 9911  
F: +44 (0) 20 8398 8032  
E-MAIL: [sales@wardray-premise.com](mailto:sales@wardray-premise.com)

**Please include the product code and the product's serial number when contacting Wardray Premise.**



## **Cleaning**

This product should be washed with a disposable cloth wrung out in a solution of warm water and detergent. The product should be rinsed with warm water and dried thoroughly with paper towels.

If required the product can be washed down with hand hot sodium hypochlorite solution up to 1% or 10000 parts per million chlorine combined with a detergent. Alternatively, follow your local guidelines or instructions.

**If any of the above give cause for concern, please contact your Wardray Premise service provider.**

## **EU Declarations of Conformity and Magnetic Testing**

Documentation is available upon request

## WARRANTY REGISTRATION FORM

**To receive full warranty cover on your equipment, please complete and return this form at your earliest convenience**

**Reply e-mail: [sales@wardray-premise.com](mailto:sales@wardray-premise.com)**

Name:	
Job Title:	
Company:	
Address:	
Telephone:	
Fax:	
Email:	
Product Code/ Description:	
Product Serial/ Reference No:	
Please enter date Product received:	

If your department currently has any other Wardray Premise Ltd. equipment, which is not already covered under a service contract, please contact [Servicing-Admin@wardray-premise.com](mailto:Servicing-Admin@wardray-premise.com) to discuss an all-inclusive package.

If you would like any further information or would like to be contacted regarding any of our products, please add your enquiry to the box below.

--