

# WARDRAY PREMISE

## XPC001 Proctology Chair



## OPERATORS MANUAL





## **Introduction**

Thank you for purchasing the XPC001 Proctology Chair. This product is manufactured and tested to the highest standards.

Our products are manufactured by Wardray Premise Ltd to BS EN ISO 13485:2016 and are CE marked.

### **Manufacturing address:**

**Wardray Premise Limited, Northern Regional Office, Unit 2,  
Springvale Works, Elland Road,  
Brighouse, West Yorkshire, HD6 2RN, UK**

To ensure that you obtain maximum benefit from your product, please take a few minutes to read the enclosed information regarding operation, service and maintenance.

If you have any problems in the meantime or would like any advice about this or any other products from the Wardray Premise range, please contact us at our **Head Office**:

**Wardray Premise Limited, Hampton Court Estate, Summer Road,  
Thames Ditton, Surrey, KT7 0SP, UK**

Tel: +44 (0) 20 8398 9911

Fax: +44 (0) 20 8398 8032

E-mail: [sales@wardray-premise.com](mailto:sales@wardray-premise.com)

### **Kindly note:**

If you modify a Wardray Premise product you will invalidate your warranty.

Unless you have our express written permission to modify a product, we transfer all liabilities for modified products to you.

### **EU Authorised Representative:**

Advena Ltd  
Tower Business Centre  
2<sup>nd</sup> Floor, Tower Street  
Swatar  
BKR 4013  
Malta

## Specification

The XPC001 Proctology Chair has been constructed from Aluminium for its strength and lightweight characteristics. The seat is manufactured from Natural Polyethylene

The chair is fixed height, but should accommodate all digital imaging machines. The armrest is fitted for patient comfort and safety, it has not been designed to be weight bearing.

This chair **must not** be used to transport patients. The chair must be positioned for the X-ray and then the patient is invited to use the chair.

A step has been fitted to allow easy patient access to the raised seat, this can be retracted once the patient is seated or when the chair is being manoeuvred.

The chair is fitted with two 125mm castors at the rear, when the chair is tilted backwards the chair can be manoeuvred easily (without the patient on the chair).

A protective collar with a lead band is provided with the chair, this when placed around the plastic bed pan (not provided) prevents scatter radiation, see Fig 1 on page 4.

Overall Height mm	Width mm	Depth mm	Seat height mm	Seat width mm	Seat depth mm	Weight Capacity
1375	545	750	750	500	480	100kg

## XPC001 Proctology Chair parts reference

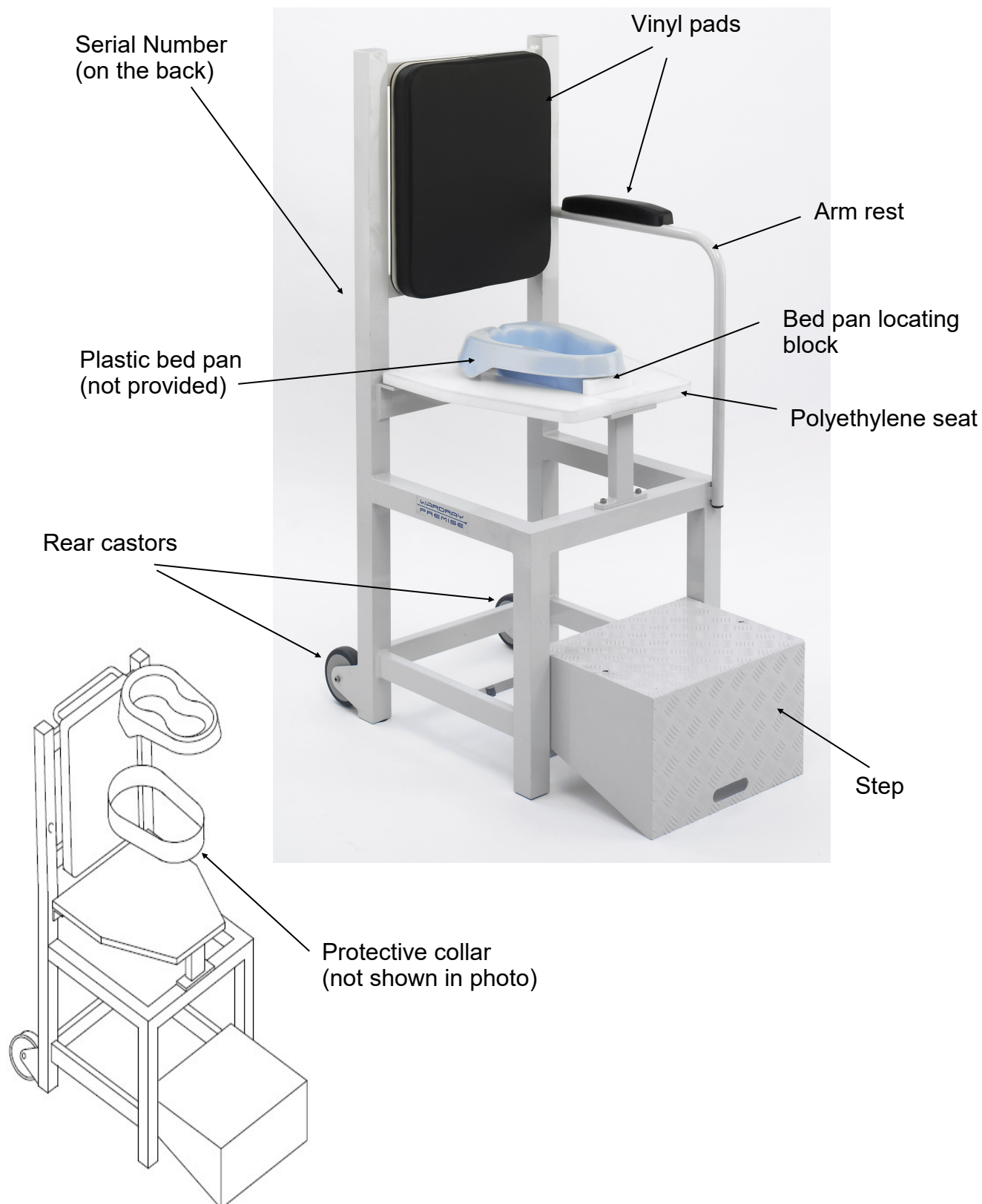


Fig 1

## **Maintenance**

It is strongly recommended that the following checks be carried out on a weekly basis by hospital personnel to ensure trouble free operation of the Chair.

- Check the vinyl pads for cracks or other obvious signs of damage or contamination. Report any damage found.
- Check rear castors for damage and free movement. Remove any debris as this can lead to a shortened life expectancy of the castor if left unattended.
- Check the step for ease of movement
- Check the Polyethylene seat for damage
- Check the Bed Pan locating blocks for damage

## **Cleaning**

This product should be washed with a disposable cloth wrung out in a solution of warm water and detergent. The product should be rinsed with warm water and dried thoroughly with paper towels.

If required the product can be washed down with hand hot sodium hypochlorite solution up to 1% or 10000 parts per million chlorine combined with a detergent. Alternatively, follow your local guidelines or instructions.

**If any of the above give cause for concern, please contact your Wardray Premise service provider.**

## **EU Declarations of Conformity**

Documentation is available upon request

## Do's and Don'ts

### **Do ensure that the following are observed**

- The Chair should only be used by properly trained staff at all times.
- The Chair must only be used for the purpose for which it was designed.
- Any problems during the warranty period should be referred to Wardray Premise, this product has a 12 month warranty period from the date of delivery and covers material defects and faulty workmanship.

### **Don't**

- Use the chair to transport a patient.
- Push the Chair over rough or uneven surfaces.
- Drop heavy or sharp objects onto the Vinyl Pad.
- Place a weight of over the equivalent of 15.7 Stone (100Kg 220lb) onto the chair.
- Spill oil or other liquid chemicals on the Vinyl Pad.
- Attempt to rectify any faults on the Chair without contacting Wardray Premise.

Please register your product with us, via the Warranty Registration Form (see page 7), for ease of contact in case of urgent upgrades or recalls.

If the unit is damaged please do not attempt to repair as this could invalidate your warranty and therefore safety, please call or email Wardray Premise Limited:

T: +44 (0) 20 8398 9911  
F: +44 (0) 20 8398 8032  
E-MAIL: [sales@wardray-premise.com](mailto:sales@wardray-premise.com)

**Please include the product code and the product's serial number when contacting Wardray Premise.**

## WARRANTY REGISTRATION FORM

**To receive full warranty cover on your equipment, please complete and return this form at your earliest convenience**

**Reply e-mail:** [sales@wardray-premise.com](mailto:sales@wardray-premise.com)

**Reply Fax:** +44 (0)20 8398 8032

Name:	
Job Title:	
Company:	
Address:	
Telephone:	
Fax:	
Email:	
Product Code/ Description:	
Product Serial/ Reference No:	
Please enter date Product received:	

If your department currently has any other Wardray Premise Ltd. equipment, which is not already covered under a service contract, please contact [Servicing-Admin@wardray-premise.com](mailto:Servicing-Admin@wardray-premise.com) to discuss an all-inclusive package.

If you would like any further information or would like to be contacted regarding any of our products, please add your enquiry to the box below.

--